

## INNOVAC COLLECTION METHOD

### Using Quick-Spot Dispenser for Newborn Screening And Dried Blood Spot Specimen Collection

This method is consistent with the CLSI (NCCLS) Blood Collection on Filter Paper For Newborn Screening Programs; Approved Standard –Fifth Edition (LA4-A5). The user should be familiar with the techniques of this standard.

- 1) Use a heparinized (**GREEN**) Quick-Draw® Adapter (IMT #15-971) assembled with a lithium heparin MiniCollect® (without gel separator) (GBO #450477). Collect the sample using this assembly attached to the Innovac® Vacuum System (follow instructions supplied with the Innovac)\*. **Fill at least 1/3 to ½ full (350-400µl), if possible.** Remove the Quick-Draw / MiniCollect® assembly from the Innovac and mix well by inverting several times. After collection, you can take your time to treat the patient before moving on to the next step.
- 2) **Remove the Quick-Draw Adapter from the MiniCollect® and discard into a biohazard container. Attach a new Quick-Spot Adapter (no straw) (IMT #15-985) onto the MiniCollect® containing the sample.** This step assures that you will dispense the entire sample from the MiniCollect®. Attach this assembly to the disk filter on the Quick-Spot Specimen Dispenser (IMT #15-675). Position the MiniCollect® in a horizontal position until you are ready to dispense sample (when in the vertical position, specimen may drip from the tip of the Quick-Spot Adapter).
- 3) When ready, rotate the MiniCollect with Q-S Adapter assembly to be vertical as shown below. Slowly and evenly press down the plunger on the Quick-Spot Dispenser and **wipe away the first drop of blood** that is dispensed onto a cotton ball or other absorbent material. **If you were unable to collect 350 - 400µl of blood, skip this step and use the first drop on the first circle.** (The first drop may not be a full 75µl)
- 4) Position the tip of the Quick-Spot Adapter over the first circle of the test paper, being **careful not to touch the test paper with the tip of the Adapter.** Hold the tip close to the circle without touching the paper (approx. 1/4”).
- 5) **Slowly (1 second) and evenly, COMPLETELY depress the Quick-Spot plunger** to drop the sample from the tip of the Quick-Spot Adapter to the sample site of the test paper. **Drop the specimen onto the center of the circle and let it flow out to the edges.** The sample may dispense in more than one small drop, but will be dispensed quickly enough so “layering” does not occur. **After dispensing the drop, release the plunger and turn your wrist to rotate the Q-S Dispenser approximately 45 degrees** (this prevents any sample from dripping out between spots).
- 6) Move to the next sample site, **rotate back to vertical**, and repeat step 5 until all circles are filled.
- 7) After dispensing the sample, remove the Quick-Spot Adapter / MiniCollect® Assembly from the Quick-Spot Specimen Dispenser and dispose in a biohazard container.
- 8) The Quick-Spot Specimen Dispenser device **is not disposable**; it is designed for 400 patients (2,000 dispenses). If it needs to be disinfected, wipe it down with a disinfectant wipe. Do not immerse.

\* Inspect the disk filter before each use. If it has blood in it, replace it with a new one (#15-501).

**NOTE: The one-way valves in the gold-colored parts are red in color and do not contain blood.**

#### \*ALTERNATE COLLECTION METHOD:

- The sample may be collected without utilizing the Quick-Draw Adapter and Innovac Vacuum System. Standard collection methods with heparin MiniCollect® (GBO #450477) may be used. Be sure to fill 1/3 to ½ full if possible. With this method, start with step 2 above.



QUICK-SPOT SPECIMEN DISPENSER

Brand name and part numbers shown are intended only to show the compatibility of this Innovative Med Tech product. Innovative Med Tech products are not associated with or endorsed by Greiner Bio-One. Innovac® and Quick-Draw® are registered trademarks of Innovative Medical Technologies, Inc. MINICOLLECT® is a Trademarks of Greiner Bio-One