



## INSTRUCTIONS FOR USE


1. Using the correct lancet or incision device is just as important as using the correct collection device. Be sure the lancet being used provides an adequate flow of blood for the sample size you are collecting. Some other considerations are to warm the site, massage the hand and finger prior to making the incision, or select a finger without calluses.
2. The Capi-Draw system works by the traditional method of capillary action not by the “scoop method”. Be sure the tube is held at a horizontal or very slight downward angle. Allow a large drop to form on the finger and place the capillary close to the drop. Let the capillary tube do the work for you; the blood will flow in immediately. Continue until the capillary tube is full. Do not scoop.

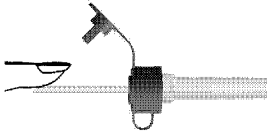

3. After filling, turn the capillary/tube unit immediately in a vertical position to allow the blood to flow into the tube. If necessary, gently tap the bottom of the collection vial on a soft surface (finger or palm) to drop the blood from the capillary into the collection vial.

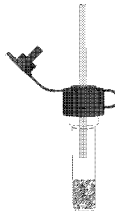

4. Remove the capillary and dispose in a biohazard container. Close the tube with the attached stopper. Mix or centrifuge blood as required. For re-opening the vessel use a slight left-turn.

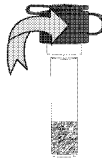

For re-closing the vessel use a slight pressure on the lid.