



Helping all people live healthy lives

# BD Microtainer™ Tubes with Microgard™ Closure Tube Guide and Order of Draw



365976  
Tube Extender



BD Vacutainer Systems  
Preanalytical Solutions  
1 Becton Drive  
Franklin Lakes, NJ 07417

Catalog #/ Closure Color	Additive	Mix by Inverting	Laboratory Use
--------------------------	----------	------------------	----------------



365974 Lavender

K<sub>2</sub> EDTA

10x

For whole blood hematology determinations. Tube inversions prevent clotting.



365965 Green

Lithium Heparin

10x

For plasma determinations in chemistry. Tube inversions prevent clotting.



365985 Mint Green



365987 Mint Green

Lithium Heparin and Gel for plasma separation

10x

For plasma determinations in chemistry. Tube inversions prevent clotting.



365992 Grey

NaF/Na<sub>2</sub> EDTA

10x

For glucose determinations. Tube inversions ensure proper mixing of additive and blood.



365967 Gold



365978 Gold

Clot Activator and Gel for serum separation

5x

For serum determinations in chemistry.



365963 Red

No additive

0x

For serum determinations in chemistry, serology and blood banking.

BD Vacutainer Technical Services: 1.800.631.0174  
BD Customer Service: 1.888.237.2762  
www.bd.com/vacutainer

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2003 BD. Made in USA 5/03 V55836-1

QUICK-DRAW ADAPTER	MICROTAINER
15-974	365974
15-900	365967 / 365978 / 365963
15-965	365965
15-980	365985 / 365987

