

INSTRUCTIONS FOR USE

For use with VacuFlo® MFR#s 607, 608.

PRODUCT DESCRIPTION:

The safety blood collection needles are assembled with protective needle caps, needle hub, needle cannula, isoprene rubber sheath, and safety shield which can be activated to cover the needle immediately following blood collection to aid in the protection against accidental needlestick injury. Models can be identified by the color coding of needle hub:

Model	607	608
Needle	21G x 1 ¼" (0.8 x 32mm)	22G x 1 ¼" (0.7 x 32mm)
Color	Deep Green	Black

INTENDED USE:

FOR SINGLE USE ONLY. Clinical use for collecting blood sample.

PRECAUTIONS

- Handled by appropriately-trained healthcare professionals.
- Handle all biologic samples and blood collection "sharps" (Lancets, needles, Luer adapters and blood collection sets) in accordance with the policies and procedures of your facility.
- Obtain appropriate medical attention in the event of any exposure to biologic samples (e.g. through a puncture injury) since samples may transmit viral hepatitis, HIV (AIDS) or other infectious diseases.
- Do not forcefully release or re-activate the safety shield after it has been activated.
- Keep hands behind needle at all times during use and disposal of the device.

CAUTIONS

- Single use only. A reuse of the product may cause harmful infections, injury or death.
- Gloves should be worn when handling blood collection needle and during venipuncture to minimize exposure hazard.
- Do not use if the package is opened or damaged, needle is defective, or the safety shield is not firmly assembled. Do not use past expiration date on the label.
- Discard all blood collection "sharps" in biohazard containers approved for their disposal.

STORAGE

Store the device in a dry, ventilated place. Avoid direct sunlight and away from the source of heat or ignition.

INSTRUCTIONS FOR USE:

Product is sterilized by ETO, non-pyrogenic and sterile in unopened or undamaged primary packaging (paper seal intact).

1. Holding both the pink safety shield and the colored needle cap, twist and remove the clear needle cap (the rubber sheath will remain on the needle)
2. While holding the pink safety shield and the colored needle cap firmly, screw tube holder onto the needle until it fits securely and the needle is fully seated onto the holder.
3. Position the pink safety shield straight back toward the tube holder.
4. Twist and pull colored needle cap straight off.
 - a. The needle bevel is always in position for venipuncture when the pink safety shield is facing up. Do Not twist or rotate the pink safety shield.
5. Perform venipuncture according to your facility's established procedure
6. Immediately after removing needle from vein, position thumb squarely on pink safety shield thumb pad and push pink safety shield forward to cover needle. An audible click may be heard. Lock shield into place and inspect. **DO NOT** attempt to engage a safety shield by pressing against a hard surface.
7. Discard immediately into an approved sharps disposal container. **DO NOT** remove needle from holder. Dispose of the needle and holder as one unit into nearest sharps container. **DO NOT REUSE.**

Made in China for:

Innovative Medical Technologies, Inc.

Omaha, NE 68144